HOJA DE ESCALPELO INFORMACIÓN

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SCALPEL BLADE INFORMATION

Caution: Federal (U.S.A.) law restricts this device to sale or on the order of a physician.

CAUTION: DO NOT BEND BLADES PRIOR TO OR DURING USE. DO NOT BEND OR APPLY PRESSURE TO BLADES WHEN CLEANING (INCLUDING STROPPING). MONITOR THE CONTROLLER FRONT PANEL TO CONFIRM BLADE INTEGRITY.

CAUTION: BLADES ARE SURGICALLY SHARP. USE STANDARD SHARPS PRECAUTIONS WHEN HANDLING.

HOW SUPPLIED

C2Dx's Scalpel Blade is provided sterile.

CAUTION: Inspect all packages for punctures or evidence of contamination prior to opening. The products will remain sterile in an undamaged, unopened package.

NOTE: If the blade becomes accidentally dislodged from the handle, turn the handle OFF, dip the blade in sterile water or saline solution to cool it to room temperature, and carefully reinsert the blade. Accurate calibration of the blade can only be achieved if the blade is at room temperature when it is inserted or reinserted into the handle. Care should be taken not to bend the blade while cleaning, insertion, or removal and subsequent reinsertion as the blade's micro-circuitry may become broken and the blade stop working. Never use any type of abrasive pad to clean the blades. The abrasives will damage the circuit and render the blade unusable.

Intended for single-use only. DO NOT RESTERILIZE.

Information on known characteristics and technical factors known to the manufacturer that could pose a risk if the Hemostatix Thermal Scalpel blades were to be re-used include, but are not necessarily limited to, the following:

Infection. Hemostatix Thermal Scalpel blades are initially sterilized using ethylene oxide gas. The manufacturer has not studied the effects of any other method of sterilization on Hemostatix Thermal Scalpel blades, nor has the manufacturer studied the effects of cleaning and re-sterilization of a used blade.

Blade Design. The blades incorporate heating and temperature-sensing micro-circuitry which provides heat for hemostasis and sensing feedback to the controller which are easily broken should the blades be bent in any attempts at reprocessing. The blades become dull with use. Any attempt to sharpen a dull blade could bend the blade and/or grind into the micro-circuitry incorporated in the blade's design rendering it useless. Re-sharpening a blade, excessive scrubbing or cleaning of a blade, or using an instrument such as a forceps to either remove a blade from or insert a blade into a handle may remove part of the non-stick coated incorporated on the blade's surface, damage the blade's micro-circuitry and/or bend the blade and subsequently damage the blade's electrical circuit.

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- HEMOSTATIX®, C2Dx, Inc. firmasinn ABD' de Tescilli Ticari Markasdır (No. 2128414) © 2022 C2Dx, Inc.
- HEMOSTATIX® is a U.S.A. Registered Trademark (#2128414) of C2Dx, Inc.
- U.S.A. (#2128414) Marque déposée de C2Dx, Inc.
- U.S.A. (#2128414) Eingetragene Marke von C2Dx, Inc
- U.S.A. (#2128414) Marchio depositato di C2Dx, Inc
- U.S.A. (#2128414) Marca registrada de C2Dx, Inc